



① How to apply vial sticker

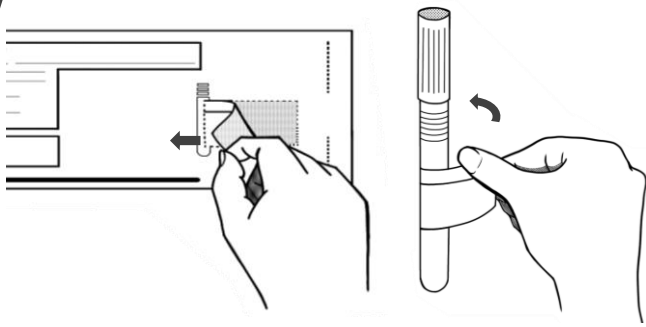

Remove the name sticker from the Request bag and rap tightly around the vial.

- ※Please dispose the vial if expiration date has passed.
- ※Please be sure to label with your name written.


② How to remove the cap

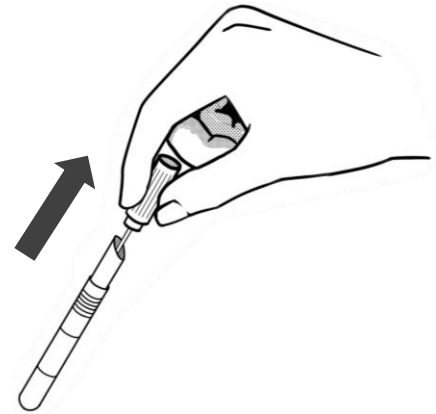

Hold the vial and pull the cap slowly.

- ※Do not discard the blue liquid in the specimen vial.


③ How to collect stool

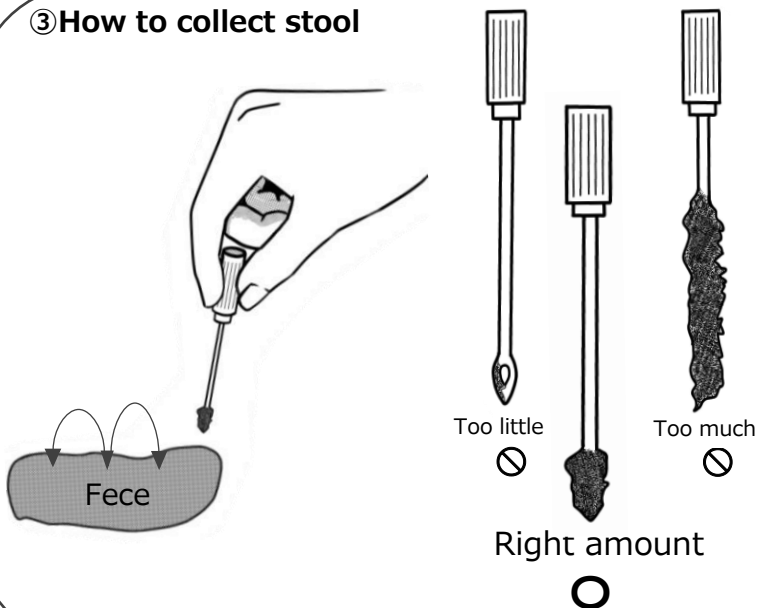

Scoop the surface of the stool as directed (correct amount needed, please do not overfill)


- ※Same amount for watery and bloody stool.
- ※Failure to add sufficient sample may lead to invalidate result.
- ※Do not insert anything but a stool sample in the vial.

④ How to close the cap

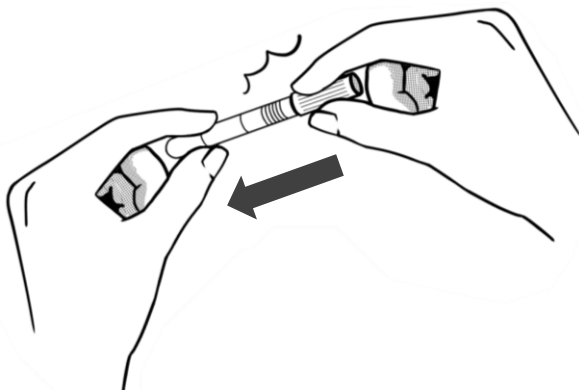

Press down the cap firmly until you hear a "click" or "snap".

⑤ How to seal the bag

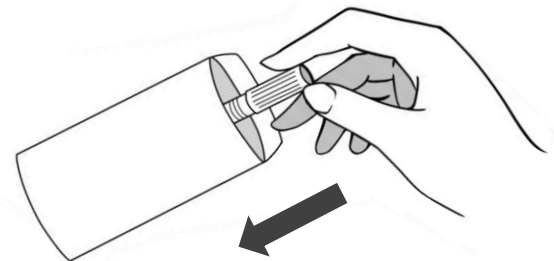

Place capped specimen vial into the paper bag and seal. Please send the Specimen vial by mail or using the transport company at room temperature.

- ※Stool sample must be received within 7 business days of collection. Accept dates : Mon~Fri(holidays not

** How to fill out the test requisition **

① Reviewing your information

Bag with your name printed

【If given written bag】

【1】 Please review all the information listed in the request bag.

Bag with no name

【If given blank bag】

【1】 Please review all the information listed in the request bag.

【2】 Write your name on the request bag and vial sticker. (2 places)

※If you fail to write your name in 2 places as directed, the result will listed as "No name".

② How to make corrections

【If it needs a

【1】 If correction needs to be made,color in the correction box and draw lines in incorrect areas. Write correct information using the red ink.

③ If you are Retesting

【Retesting request】

【1】 If your are Retesting, Please write "RETEST" on the right side of the bag in red writing.